COVID-19

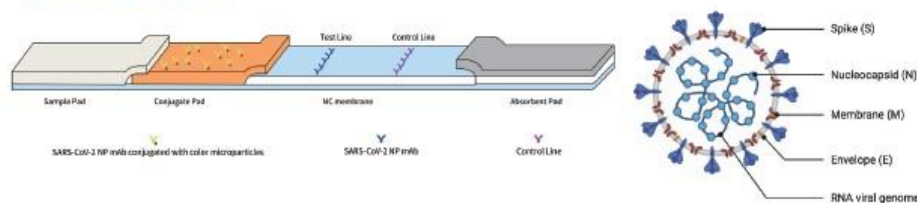
Corona Virus (COVID-19) Antigen Rapid Test (Colloidal Gold)

- China Medical Device Exportation Whitelist
- ISO 13485: 2016
- CE

● Product information

Model		Packing specification	
HYT-G01-01	Nasopharyngeal swab	30 Tests/ box	1 Test/ box
HYT-G01-02	Oropharyngeal swab	30 Tests/ box	1 Test/ box
HYT-G01-03	Saliva	20 Tests/ box	1 Test/ box

● Test principle

The COVID-19 Antigen Rapid Test is a lateral flow immunoassay based on the principle of the double-antibody sandwich technique. A colored test line (T) would be visible in the result window, if COVID-19 antigens are present in the specimen. Absence of the T line suggests a negative result.

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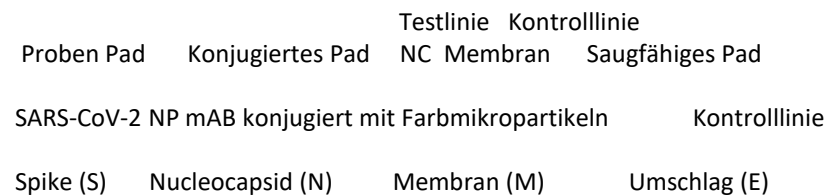
Product advantage

Facilitate higher-volume testing and offer rapid results;
SARS-CoV-2 is detectable in saliva, which provides a less invasive and more scalable specimen collection approach.

Easy to operate

No specialist equipment required
Can be used at any convenient places and time

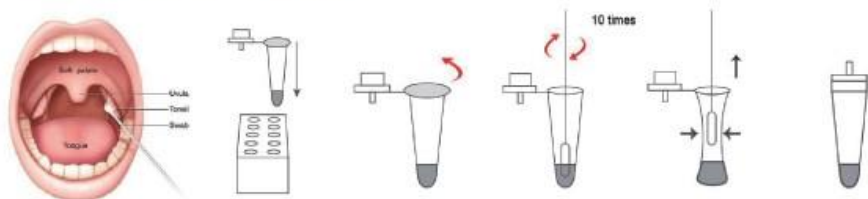
Contain all necessary materials
Get quick results within 15 minutes

Test Procedure

Inadequate specimen collection or improper specimen handling may yield a false result.
Prior to collecting the swab samples, the patient should be instructed to blow their nose.
HYT-G01-01 / Nasopharyngeal swab Nasopharyngeal Abstrich

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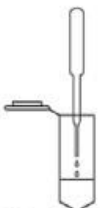
After gargle, spit saliva into the sample collection cup

add 100uL (About 3 drops) saliva to the sample dilution tube

cover the lid and shake the bottle 5 times 5-mal

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